HemoClin[®]

gel

HemoClin[®] Gel

- Instant cooling relief from haemorrhoids and anal discomforts (i.e. itching, irritation, burning, sensitivity)
- Prevents haemorrhoids and fissures
- Treats and prevents anal discomforts
- Conditions the skin and supports the natural healing process

Please read the entire Instructions for Use carefully before you start using HemoClin[®]. Do not throw this leaflet away. If you have any questions, or if you are not sure about anything, please ask your doctor or pharmacist.

Contents of this leaflet:

1. What is HemoClin[®] used for?
2. What does HemoClin[®] contain?
3. What do the active ingredients in this product do?
4. What you should know before using HemoClin[®]?
5. When and how should you use HemoClin[®]?
6. Storage
7. Contents
8. Contact details

1. What is HemoClin[®] used for?

HemoClin[®] treats anal discomforts and is used to relieve the symptoms of irritation, itching, burning and sensitivity associated with haemorrhoids. HemoClin[®] both prevents and treats anal discomforts. Haemorrhoids and other anal discomforts are frequently found during pregnancy and after childbirth.

2. What does HemoClin[®] contain?

What is in the gel?

HemoClin[®] contains 37g of a gel based on the patented bio-active 2QR-complex, which is derived from the Aloe Barbadensis plant. This product has no fragrance and no ingredients of animal origin.

3. What do the active ingredients in this product do?

HemoClin[®] is effective in the prevention and treatment of anal discomforts such as itching, irritation, burning and sensitivity. The bio-active gel offers rapid relief from haemorrhoids, fissures and anal discomforts. The bio-active ingredient, named 2QR, neutralizes harmful microbes, supports the natural healing process and conditions the tissue. HemoClin[®] also provides lubrication to facilitate bowel movements.

Ingredients:

Aqua, Aloe Barbadensis Gel, Galactoarabinan Polyglucuronic Acid Crosspolymer*, PEG-8, Xanthan Gum, Betaine, Laureth-9, Phenoxylethanol, Ethylhexyl-glycerin, Sodium Hydroxide, Potassium phosphate. * 2QR

4. What you should know before using HemoClin[®]?

HemoClin[®] does not contain any harsh-chemicals. No side effects are known during the usage of HemoClin[®].

Please talk to your doctor first if:

You continually suffer from haemorrhoids or experience excessive bleeding.

Use of other medications:

No interference with other medications is known.

5. When and how should you use HemoClin[®]?

For treatment of anal discomforts (i.e. irritation, itching, burning, sensitivity):

Apply in and around the affected area using the gel at least twice per day in ample quantity for a period of two weeks or whenever relief is desired. More frequent application increases the effectiveness.

For prevention of (internal and external) haemorrhoids and fissures:

Apply in and around the anus before and after each bowel movement. Place a small amount of the HemoClin[®] gel on the finger. Apply in and around the anal canal whenever relief is needed. Securely Close tube after use to keep the gel from drying out.

HemoClin[®] should not be used for longer than 30 continuous days in one treatment period.

Do not use this product if you are sensitive to one or more of the ingredients.

6. Storage

HemoClin[®] should be used at room temperature. Store in a safe and cool place. Do not use after the expiry date shown on the top of the tube and on the packaging.

7. Contents

- A 37g Tube containing HemoClin Gel
- Information leaflet
- Canule

Warning:

Keep out of the reach of children.

Manufacturer:

You Medical, YouMedical B.V.,
Rijnsburgstraat 9-11, 1059 AT Amsterdam, The Netherlands.

Distributor:

Health and Beauty Care Vietnam Co., Ltd,
29 - 31 Nguyen Hien Le St, TanBinh Dist,
HCM City, Vietnam.

For more information about HemoClin, please call us:
08-62962262

All enquiries about the product should be sent to the distributor.

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